Advanced Instruments

Calibration standards

REF	3MA000	Ten 2 mL ampules of 0 mOsm/kg H ₂ O
	3MA005	Ten 2 mL ampules of 50 mOsm/kg H ₂ O
	3LA011	Ten 5 mL ampules of 100 mOsm/kg H ₂ O
	3MA020	Ten 2 mL ampules of 200 mOsm/kg H ₂ O
	3MA003	Ten 2 mL ampules of 300 mOsm/kg H ₂ O
	3MA040	Ten 2 mL ampules of 400 mOsm/kg H ₂ O
	3LA051	Ten 5 mL ampules of 500 mOsm/kg H ₂ O
	3MA085	Ten 2 mL ampules of 850 mOsm/kg H ₂ O
	3LA091	Ten 5 mL ampules of 900 mOsm/kg H ₂ O
	3MA100	Ten 2 mL ampules of 1000 mOsm/kg H ₂ O
	3LA151	Ten 5 mL ampules of 1500 mOsm/kg H ₂ O
	3MA200	Ten 2 mL ampules of 2000 mOsm/kg H ₂ O
	3LA201	Ten 5 mL ampules of 2000 mOsm/kg H ₂ O
	3LA301	Ten 5 mL ampules of 3000 mOsm/kg H ₂ O
	3MA400	Ten 2 mL ampules of 4000 mOsm/kg H ₂ O

Intended use

Calibration standards are used to calibrate and/or check the performance of your osmometer. For verification, please choose the appropriate standard based on the sample range and the osmometer specifications. Refer to the instrument User's Guide for calibration instructions.

Ingredients

These standards are prepared from salts. 2-Hydroxybiphenyl Sodium Salt Tetrahydrate, 0.01%, is added as a preservative.

Warning

Carefully snap off the top of the ampule. There may be sharp edges at breakpoint, and could result in injury if caution is not taken.

Instructions for use

- Carefully snap off the top of the ampule.
- Obtain samples with clean sample tips. The sample size will be based on the osmometer that is being used. Refer to User's Guide for more information about sample size and testing procedure.

Storage and handling

Follow these storage and handling guidelines for optimal results:

- Dispose of any opened ampules at the end of the testing day.
- Do not freeze.

	Storage	Stability
Unopened	2-30 °C (36-86 °F)	Refer to expiration date stamped on the box

Limitations

If there is visible evidence of microbial growth in the ampules, do not use the calibration standards. Erroneous results can occur from adverse shipping and/or storage conditions, use of expired materials, or sample handling errors.

Interpreting results

Please visit aicompanies.com/expectedranges for expected values and expected ranges based on your osmometer model.

NOTE: If results fall outside the total expected range, it may indicate unsatisfactory calibration, operator error, contamination of reagents, or faulty performance of equipment.

Disposing of materials

Handle this product according to established good laboratory practices, using appropriate precautions. Dispose of materials according to your institution's practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state and local requirements.

For sales and service

Contact your Advanced Instruments Distributor.

For *In Vitro* Diagnostic Use

Consult Instructions for Use
参見説明
Viz. pokyny
Les Brugsanvisningen
Συμβουλευτείτε τις Οδηγίες Χρήσης
Zie instructies
Consulter le mode d'emploi
Bedienungsanleitung lesen
Leggere le istruzioni per l'uso
使用説明書をよくご覧ください
설명서 참조
Consultar instruções
См. инструкции
Vid pokyny
Consulte las instrucciones de uso
Läs bruksanvisningarna
Tallimata Bakin

IVD
For *In vitro* Diagnostic Use
用于试管诊断
Pro *in vitro* diagnostické použití
Til diagnostisk brug *in vitro*
Για *in vitro* *in vitro*
Voor *in-vitro* diagnostiek
Pour un diagnostic *in vitro*
Zur *In-vitro*-Diagnose
Da utilizzarsi per diagnostica *in vitro*
体外診断用
제외 진단용
Para uso em diagnóstico *in vitro*
Для диагностики
Pre *in vitro* diagnostické použitie
Para diagnósticos *in vitro*
För *in vitro*-diagnostik
In Vitro Tanılama Kullanımı İçindir

LOT
Lot Number
批号
Číslo šarže
Lotnummer
Αριθμός Παρτίδας
Partijnummer
Número de lot
Losnummer
Numero lotto
ロット番号
로트 번호
Número do Lote
Номер партии
Číslo šarže
Número de lote
Partinummer
Lot numarası.

Temperature Limit
温度限制
Teplotní limit
Temperaturgrænse
Όριο Θερμοκρασίας
Temperatuurlimiet
Limite de température
Temperaturgrenze
Limite di temperatura
限界温度
온도 한계
Limite de temperatura
Предельная температура
Teplotný limit
Limite de temperatura
Temperaturgräns
Sıcaklık Sınırı

Use By
過期日期
Použitelné do; Expirační doba
Anvendes inden
Χρήση έως
Gebruiken voor; Vervaldatum
Date limite de conservation
Verfalldatum
Scadenza
使用期限
유효기간까지 사용
Prazo de validade
Дата окончания срока действия
Použitelné do; doba platnosti
Fecha de vencimiento
Använd senast
Son Kullanım Tarihi

EC REP
WMDE B.V.
Bergerweg 18
6085 AT Horn
The Netherlands

REF
Catalog Number
零件编号
Číslo dílu
Deelnummer
Αριθμός Καταλόγου
Onderdeelnummer
Référence
Best.-Nr.
Numero catalogo
カタログ番号
부품 번호
Número de Produto
Обозначение
Číslo dielu
Número de catálogo
Katalognummer
Parça Numarası

CE
European Conformity
符合歐洲標準
Evropská shoda
Europæisk konformitet
Ευρωπαϊκή Συμμόρφωση
Europese conformiteit
Conformité européenne
Konformität mit EU-Normen
Conformità europea
ヨーロッパ規格
Conformidade Europeia
Отвечает нормативам
Európska zhoda
Conformidad europea
Uppfyllande av europeiska krav
Avrupa Uyumu

EC REP
Authorized Representative
授权的代表
Zplnomocněný zástupce
Autoriseret repræsentant
Εξουσιοδοτημένος
Erkend vertegenwoordiger
Représentant autorisé
Autorisierter Vertreter
Rappresentante autorizzato
正規代理店
승인 대리점
Representante Autorizado
Ответственный представитель
Zplnomocnený zástupca
Representante autorizado
Auktoriserad representant
Yetkili Temsilci a

Manufacturer
製造商
Výrobce
Producent
Κατασκευαστής
Fabrikant
Fabricant
Hersteller
Fabricante
製造者
제조사
Fabricante
Изготовитель
Výrobca
Fabricante
Tillverkare
Üretici Firma

Use By
過期日期
Použitelné do; Expirační doba
Anvendes inden
Χρήση έως
Gebruiken voor; Vervaldatum
Date limite de conservation
Verfalldatum
Scadenza
使用期限
유효기간까지 사용
Prazo de validade
Дата окончания срока действия
Použitelné do; doba platnosti
Fecha de vencimiento
Använd senast
Son Kullanım Tarihi

ADVANCED INSTRUMENTS
Advanced Instruments, LLC
Two Technology Way
Norwood, Massachusetts 02062, USA
781-320-9000 800-225-4034
aicompanies.com

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