Advanced Instruments

Osmolality Linearity Set

REF	3LA028	Two 5 mL ampules of 100 mOsm/kg H ₂ O
		Two 5 mL ampules of 500 mOsm/kg H ₂ O
		Two 5 mL ampules of 900 mOsm/kg H ₂ O
		Two 5 mL ampules of 1500 mOsm/kg H ₂ O
		Two 5 mL ampules of 2000 mOsm/kg H ₂ O

Intended use

The Osmolality Linearity Set is designed to help laboratories easily monitor osmometer performance specifications.

- Assess and document method performance standards.
- Test Linearity Set after performing calibration.
- Verify reportable range of sample results.
- Strengthen your laboratory's Quality Control program by using products made specifically for your osmometer.
- For clinical laboratories, fulfill CLIA requirements for verifying the reportable range of a laboratory method.

Ingredients

These standards are prepared from salts. 2-Hydroxybiphenyl Sodium Salt Tetrahydrate, 0.01%, is added as a preservative.

Warning

Carefully snap off the top of the ampule. There may be sharp edges at breakpoint, and could result in injury if caution is not taken.

Instructions for use

- Carefully snap off the top of the ampule.
- Obtain samples with clean sample tips. The sample size will be based on the osmometer that is being used. Refer to User's Guide for more information about sample size and testing procedure.

Storage and handling

Follow these storage and handling guidelines for optimal results:

- Dispose of any opened ampules at the end of the testing day.
- Do not freeze.

	Storage	Stability
Unopened	2-30 °C (36-86 °F)	Refer to expiration date on the box

Limitations

If there is visible evidence of microbial growth in the ampules, do not use the Linearity Set. Erroneous results can occur from adverse shipping and/or storage conditions, use of expired materials, or sample handling errors.

Interpreting results

Please visit aicompanies.com/expectedranges for expected values and expected ranges based on your osmometer model.

NOTE: If results fall outside the total expected range, it may indicate unsatisfactory calibration, operator error, contamination of reagents, or faulty performance of equipment.

Disposing of materials

Handle this product according to established good laboratory practices, using appropriate precautions. Dispose of materials according to your institution's practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state and local requirements.

For sales and service

Contact your Advanced Instruments Distributor.

For *In Vitro* Diagnostic Use

Consult Instructions for Use
 参見說明
 Viz. pokyny
 Las Brugsanvisningen
 Συμβουλευτείτε τις Οδηγίες Χρήσης
 Zie instructies
 Consulter le mode d'emploi
 Bedienungsanleitung lesen
 Leggere le istruzioni per l'uso
 使用説明書を正確に読んで
 설명서 참조
 Consultar instruções
 См. инструкции
 Vid pokyny
 Consulte las instrucciones de uso
 Läs bruksanvisningarna
 Talimata Bakin

For *In vitro* Diagnostic Use
 用于试管诊断
 Pro *in vitro* diagnostické použití
 Til diagnostisk brug *in vitro*
 Για διάγνωση *in vitro*
 Voor *in-vitro* diagnostiek
 Pour un diagnostic *in vitro*
 Zur *In-vitro*-Diagnose
 Da utilizzarsi per diagnostica *in vitro*
 体外診断用
 Para uso em diagnóstico *in vitro*
 Для диагностики
 Pre *in vitro* diagnostické použitie
 Para diagnósticos *in vitro*
 För *In vitro*-diagnostik
In Vitro Tanımlama Kullanımı İçindir

Lot Number
 批号
 Číslo šarže
 Lotnummer
 Αριθμός Παρτίδας
 Partijnummer
 Numéro de lot
 Losnummer
 Numero lotto
 ロット番号
 로트 번호
 Número do Lote
 Номер партии
 Číslo šarže
 Número de lote
 Partinummer
 Lot numarası.

Temperature Limit
 温度限制
 Teplotní limit
 Temperaturgrænse
 Όριο Θερμοκρασίας
 Temperatuurlimiet
 Limite de température
 Temperaturgrenze
 Limite di temperatura
 限界温度
 온도 한계
 Limite de temperatura
 Предельная температура
 Teplotný limit
 Limite de temperatura
 Temperaturgräns
 Sıcaklık Sınırı it

Use By
 过期日期
 Použitelné do; Expirační doba
 Anvendes inden
 Χρήση Έως
 Gebruiken voor; Vervaldatum
 Date limite de conservation
 Verfalldatum
 Scadenza
 使用期限
 유효기한까지 사용
 Prazo de validade
 Дата окончания срока действия
 Použitelné do; doba platnosti
 Fecha de vencimiento
 Använd senast
 Son Kullanım Tarihi

Catalog Number
 零件编号
 Číslo dílu
 Delnummer
 Αριθμός Καταλόγου
 Onderdeelnummer
 Référence
 Best.-Nr.
 Numero catalogo
 カatalog番号
 부품 번호
 Número de Produto
 Обозначение
 Číslo dielu
 Número de catálogo
 Katalognummer
 Parça Numarası

European Conformity
 符合歐洲標準
 Evropská shoda
 Europeisk konformitet
 Ευρωπαϊκή Συμμόρφωση
 Europese conformiteit
 Conformité européenne
 Konformität mit EU-Normen
 Conformität europea
 コーロツク規格
 유럽 인증
 Conformidade Europeia
 Отвечает нормативам
 Európska zhoda
 Conformidad europea
 Uppfyllyande av europeiska krav
 Avrupa Uyumu

Authorized Representative
 授权的代表
 Zplnomocnený zástupce
 Autoriseret repræsentant
 Εξουσιοδοτημένος
 Erkend vertegenwoordiger
 Représentant autorisé
 Autorisierter Vertreter
 Rappresentante autorizzato
 正規代理店
 승인 대리점
 Reprezentante Autorizado
 Ответственный представитель
 Spĺnomocnený zástupca
 Representante autorizado
 Auktoriserad representant
 Yetkili Temsilci a

Manufacturer
 製造商
 Výrobce
 Producent
 Κατασκευαστής
 Fabrikant
 Fabricant
 Hersteller
 Fabbricante
 製造廠
 제조업체
 Fabricante
 Изготовитель
 Výrobca
 Fabricante
 Tilverkare
 Üretici Firma

WMDE B.V.
 Bergerweg 18
 6085 AT Horn, NLD

Advanced Instruments, LLC
 Two Technology Way
 Norwood, Massachusetts 02062, USA
 800-225-4034 781-320-9000
www.aicompanies.com

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