Advanced Instruments

OsmoTECH® HT

Calibration and Verification Standard Set

REF		
3MA000	Two 2 mL ampules of 0 mOsm/kg H ₂ O	
3MA003	Two 2 mL ampules of 300 mOsm/kg H ₂ O	
3MA085	Two 2 mL ampules of 850 mOsm/kg H ₂ O	
3MA200	Two 2 mL ampules of 2000 mOsm/kg H ₂ O	
3MA029	Two 2 mL ampules of 290 mOsm/kg H ₂ O	

Intended use

Calibration and Verification standards are used to calibrate and verify calibration on your OsmoTECH HT Plate Based Micro-osmometer. Refer to the instrument User's Guide for calibration instructions.

Ingredients

These standards are prepared from salts. 2-Hydroxybiphenyl Sodium Salt Tetrahydrate, 0.01%, is added as a preservative.

Warning

Carefully snap off the top of the ampule. There may be sharp edges at breakpoint, and could result in injury if caution is not taken.

Instructions for use

- Carefully snap off the top of the ampule.
- Load the ampules in the calibration plate, follow the onscreen instructions to perform calibration.

Storage and handling

Follow these storage and handling guidelines for optimal results:

- Dispose of any opened ampules at the end of the testing day.
- Do not freeze.

	Storage	Stability
Unopened	2 - 30 °C (36 - 86 °F)	Refer to expiration date stamped on the box

Limitations

If there is visible evidence of microbial growth in the ampules, do not use the calibration standards. Erroneous results can occur from adverse shipping and/or storage conditions, use of expired materials, or sample handling errors.

Do not use the product if the primary packaging has been damaged. Discard the product immediately.

Interpreting results

Please visit aicompanies.com/expectedranges for expected values and expected ranges based on your osmometer model.

NOTE: If results fall outside the total expected range, it may indicate unsatisfactory calibration, operator error, contamination of reagents, or faulty performance of equipment.

Disposing of materials

Handle this product according to established good laboratory practices, using appropriate precautions. Dispose of materials according to your institution's practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state and local requirements.

For sales and service

Contact your Advanced Instruments Distributor.

For In Vitro Diagnostic Use

Consult Instructions for Use
参見説明
Consulter le mode d'emploi
Bedienungsanleitung lesen
Leggere le istruzioni per l'uso
使用説明書をご覧ください
설명서 참조
Consultar instruções
См. инструкции пользователя
Consulte las instrucciones de uso
Kullanım Talimatlarına Bakın

Temperature Limit
温度限制
Limite de température
Temperaturgrenze
Limite di temperatura
限界温度
온도 한계
Limite de temperatura
Предельная температура
Limite de temperatura
Sıcaklık Sınırı

Authorized Representative
授权的代表
Représentant autorisé
Autorisierter Vertreter
Rappresentante autorizzato
正規代理店
공인 대리점
Representante Autorizado
Ответственный представитель
Representante autorizado
Yetkilili Temsilci

For In vitro Diagnostic Use
用于试验诊断
Pour un diagnostic in vitro
Zur In-vitro-Diagnose
Da utilizzarsi per diagnostica in vitro
体外診断用
재외 진단용
Para uso em diagnóstico in vitro
Para diagnósticos in vitro
Для диагностики in vitro
In Vitro Tanımlama Kullanımı İçindir

Catalog Number
零件編號
Onderdeelnummer
Référence
Best-Nr.
Numero catalogo
カタログ番号
부품 번호
Número de Produto
Обозначение
Katalog Numarası

Manufacturer
製造商
Fabricant
Hersteller
Fabbricante
製造者
제조업체
Fabricante
Изготовитель
Fabricante
Üretici Firma

Lot Number
批号
Numéro de lot
Losnummer
Numero lotto
ロット番号
로트 번호
Número do Lote
Номер партии
Número de lote
Lot numarası

European Conformity
符合欧洲标准
Conformité européenne
Konformität mit EU-Normen
Conformità europea
ヨーロッパ規格
유럽 인증
Conformidade Europeia
Отвечает нормативам ЕС
Conformidad europea
Avrupa Uygunluğu

Use By
过期日期
Date limite de conservation
Verfaldatum
Scadenza
使用期限
유효기간까지 사용
Prazo de validade
Дата окончания срока действия
Fecha de vencimiento
Son Kullanım Tarihi

WMDE B.V.
Bergerweg 18
6085 AT Horn, NLD

Advanced Instruments, LLC
Two Technology Way
Norwood, Massachusetts 02062, USA
800-225-4034 781-320-9000
www.aicompanies.com

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